

SWISSLOG CUSTOMER SUPPORT BULLETINS



6-INCH ZIP N' FOLD™ CONTAINMENT POUCHES

Zip N' Fold pouches provide maximum protection from spills, whether transporting lab specimens or pharmaceuticals in a pneumatic tube system (PTS).

The "Clean Pouch" is clearly marked in green for the transport of pharmaceuticals and other non-biohazard materials.

The red biohazard identification label on the Biohazard pouch meets the strictest OSHA standards.

Ordering Information for the Biohazard Pouch

The biohazard pouch is available with or without the vacutainer insert and is sold in sets of five.
Pouch thickness: 12-mil thick vinyl pouch, 6-mil thick vinyl insert; Closure: Ziplock and Velcro tab pouch.

Five Large Biohazard Pouches with Inserts	(10.5" x 14")	076-1319-01
Five Large Biohazard Pouches w/o Inserts		076-1319-03
Five Small Biohazard Pouches with Inserts	(10.5" x 10.5")	076-1319-02
Five Small Biohazard Pouches without Inserts		076-1319-04
Five Inserts		076-1329-01

Ordering Information for the Clean Pouch

Pouch thickness: 12-mil thick vinyl pouch, 6-mil vinyl insert; Closure: Ziplock and Velcro tab pouch.
(IV pouch is Ziplock closure only)

Single Large Clean Pouch	076-1361-01
Single Small Clean Pouch	076-1362-01
Five IV Clean Pouches	076-1366-03

Zip N' Fold Instructions

Insert primary specimen container in pouch. Squeeze out excess air and seal Ziplock. Roll the top down to attach the Velcro strips. Place pouch in the transport system carrier.

Cleaning Instructions: Sterilize with ethylene oxide gas, 10% bleach, or other equivalent disinfectant.

Due to ongoing improvement programs, Swisslog reserves the right of production or design change without notice or obligation.

For More Information in North America:

Swisslog Customer Support
Email: orderentry@swisslog.com
USA: (800) 396-9666 | Fax: (303) 373-7871
Canada: (877) 294-2831 | Fax: (905) 629-2799

©2008 Swisslog
CM# CS-604

WWW.SWISSLOG.COM

swisslog