

AutoBox is a buffer module that automatically interfaces with the packaging component (PillPicker) to expedite the loading and unloading of canisters (PillBoxes). AutoBox automatically stages up to twelve PillBoxes for bar-coded unit dose packaging.

Operation

Once the operator inserts the PillBoxes into AutoBox, stock replenishment orders are automatically fulfilled. AutoBox transfers one of up to twelve PillBoxes into the PillPicker. The PillBox lid is removed and packaging proceeds for that medication. When packaging is complete, the PillBox lid is automatically replaced and locked, and the PillBox is automatically discharged from PillPicker. The next PillBox is then automatically inserted. This process continues until the packaging order is complete.





Features and Benefits

- Fast, efficient, and accurate packaging is provided through automatic interface to PillPicker
- No human interaction required, saving valuable pharmacy staff time
- Production is optimized through twelve canister staging on the carousel
- Compact footprint saves valuable floor space in the pharmacy



Packaging

AutoBox Specifications

Power requirements: supplied from PillPicker
Compressed air: supplied from PillPicker
Weight: 375 lbs (170 Kg)

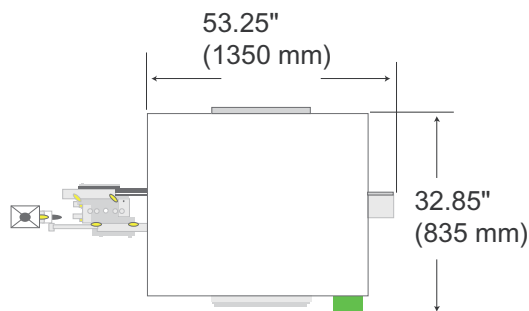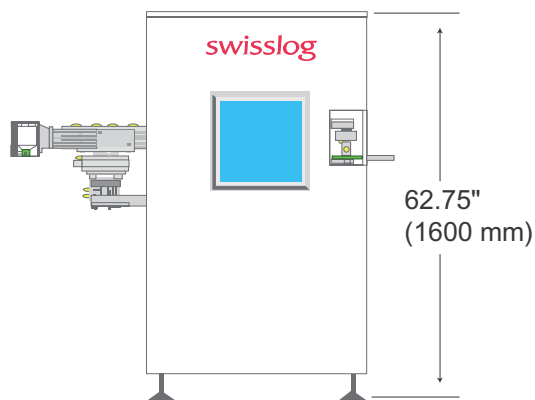

AutoBox, AutoPhial and PillPicker

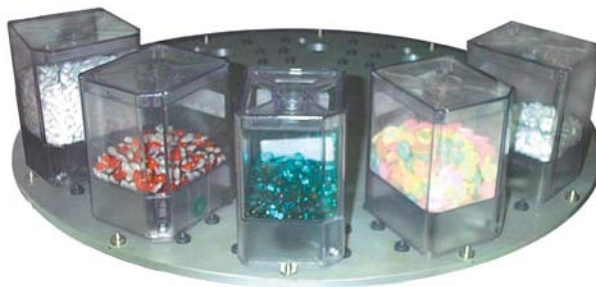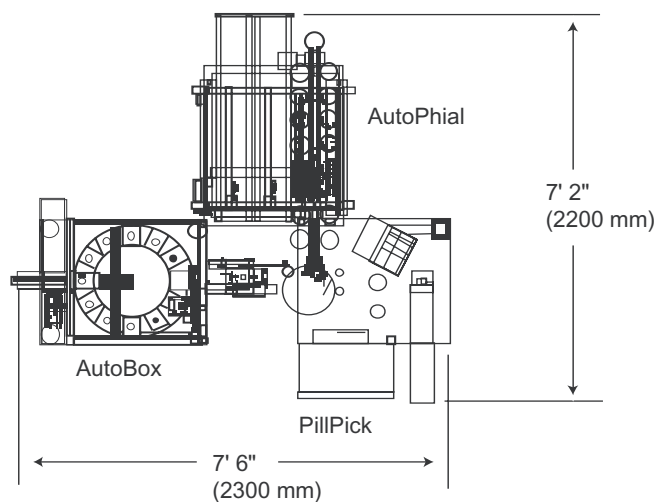

Due to ongoing improvement programs, Swisslog reserves the right of production or design change without notice or obligation.

For More Information in North America:

Email: healthcare@swisslog.com

USA: 800-764-0300; 303-373-7883

Canada: 877-294-2831; 905-629-2400

For more information about Healthcare Solutions, visit: www.swisslog.com

swisslog

© 2005 Swisslog
CM #1153